

Swan

User information appropriate for use by people with visual, reading or cognitive disability The latest version of all instructions are always available on the R82 website support.R82.org and can be printed in larger sizes

Information available in pictogram The instructions are made as pictograms

Intended use and intended environment including standard options The Swan is a toilet and bathing chair. The user is guided into the seat. A foot support, hip belt and a vest will almost always be applied, and the person can be supported by side supports and a head support. The product is for indoor use only. The target group for this device is disabled children with cerebral palsy (CP). Also, children with muscular dystrophy, spina bifida, spinal cord injuries and neurological impairments can be subject to the device. The product is suitable for children with GMFCS level 3-5.

Maintenance instructions Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

Frequency	Description
Before every use	- Disinfect the product. please check the disinfection section in this User Guide for more information
After every use	- Wipe the product dry
Daily	- Use a cloth with warm water and a mild detergent to clean the product - Visual inspection. Repair or replace any damaged or worn-out parts - Visual inspection. Repair or replace any damaged or worn-out parts - Check all fittings and buckles for signs of worn out parts - Check that all castors are moving freely and that all wheel locks/brakes are easily activated - Ensure that all fasteners are in place and fitted correctly - Be aware of any MAX mark indications before adjustment
Monthly	- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
Annually	- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

Cleaning description This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

Before washing
Remove the following items and clean them separately: Cushioning & Accessories

Swan	
Disinfection description	The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.
Dimensions (Incl. max load)	http://www.R82.org - Select product or click here
Mass	N/A
Used in combination with other products	N/A
Warning about dangerous combinations	The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.
List of accessories, detachable parts and materials	http://www.R82.org - Select product or click here
Programmable controller information	N/A
Operator control adjustments	Manual
Folding and dismantling	N/A
Transport instructions (unoccupied)	Land and/or air
Sound power level	N/A
Turning diameter	NA
Forces on the wall	N/A
Electromagnetic environment warning	N/A
Electromagnetic influence warning	N/A
Life-time	The lifetime of the product in normal use is 5 years